VistaScan

Speicherfolien Plus

Image Plates Plus

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Placas radiográficas Plus

Film ai fosfori Plus

Speicherfolien Plus

Image Plates Plus

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EN General Information

⚠ **Image plates are toxic. Always use intraoral image plates in a light protection cover.**

Image plate

- should be erased before every new application.
- are as flexible as X-ray film. Nevertheless, handle with care. Do not bend, exert pressure, scratch or make them dirty. Do not store in damp or hot conditions.
- protect from direct sunlight and ultraviolet beams.
- should be shielded from any X-ray radiation when storing in an X-ray room.
- must be replaced if there is any sign of damage (e.g. protective layer is damaged, or visible scratches) which could interfere with diagnosis.
- will be replaced by Dürr Dental pro rata if showing signs of production or packing damage We ask that should there be any reason for complaint that this is made 7 within seven days of receipt of goods.
- can be exposed, scanned and deleted indefinitely as long as they are not mechanically damaged.

📘 See also Installation and Operating Instructions VistaScan

Storage and transport

Temperature (C°)..... <33
 Relative humidity (%)..... <80

Use of the Appliance

Temperature (C°)..... 18 - 45
 Relative humidity (%)..... <80

Image Plates Plus ID

📘 Before every use, check whether the marker is attached correctly. If the marker does not adhere, do not use the image plate again.

Description

The marker **(1)** on the inactive side of the image plate indicates if the image plate has been exposed from the wrong side contrary to the intended use.

A hexadecimal code **(2)** is visible in the X-ray picture and on the image plate. The code ensures a reliable correlation between image plate and image.

Detection

If the shadow of the marker **(1)** is visible in the X-ray image, the image plate was exposed from the inactive side contrary to the intended use (cf. 3).

The image can be corrected by mirroring in the software.

If a diagnosis is not possible in the area of the marker, the X-ray should be repeated.

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