

Product information

VistaScan

image plate cleaning wipe

Characteristics

VistaScan image plate cleaning wipes are special cleaning wipes with a disinfectant effect and excellent material compatibility. They have been specially formulated for the VistaScan image plate, and clean thoroughly, while at the same time protecting the value of dental equipment thanks to their proven material compatibility.


Chemical composition

VistaScan image plate cleaning wipes contain 4 g 2-propanol.

Microbiological effectiveness

VistaScan image plate cleaning wipes have a bactericidal, tuberculocidal, and fungicidal effect¹⁾, and a limited virucidal effectiveness²⁾ (enveloped viruses such as vaccinia viruses including HBV, HCV and HIV).

Tested in accordance with EN 13727, EN 13624, EN 14348, EN 14476.

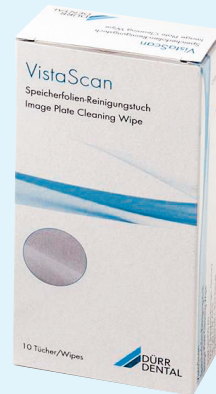

VistaScan features in brief

- Image plate cleaning wipe with disinfectant effect and excellent material compatibility.
- Specially formulated for the VistaScan image plate.
- Broad effective spectrum: bactericidal, tuberculocidal, fungicidal, limited virucidal effectiveness (enveloped viruses such as vaccinia viruses including HBV, HCV, HIV).
- Exposure time only 15 seconds.
- Tested in accordance with the European standards EN 13727, EN 13624, EN 14348 and EN 14476.
- Protects the value thanks to its tested material compatibility.

COMPRESSED AIR
SUCTION
IMAGING
DENTAL CARE
HYGIENE

Any information on our products is based on detailed internal and external scientific examinations, and recommendations are given to the best of our knowledge. Liability is judged in accordance with our General Terms of Sale and Delivery which are available on demand at any time.

Product information

VistaScan

image plate cleaning wipe

Directions for use

The VistaScan image plates should be cleaned only with VistaScan image plate cleaning wipes. Carefully wipe the image plate with the cleaning wipe and allow to dry. Only clean intact image plates.

Environmental behaviour

The active ingredient solution of the VistaScan image plate cleaning wipe is easily biodegradable. The packing is made of polyethylene and can therefore be exploited both materially and thermally.

Shelf life

4 years

Packaging units

10 wipes per folding box
10 folding boxes per carton

Storage

Store preparation in a cool place, but not below 5°C.

Caution

Flash point 25°C according to DIN 51755.
Max. overall application 50 ml/m².

Application	Time
Bacteria, fungi¹⁾	15 sec.
Tb bacteria	15 sec.
Vaccinia viruses including HBV, HCV and HIV²⁾	30 sec.

1) Tested on *Candida albicans*.

2) In accordance with RKI recommendation (Bundesgesundheitsbl. 47, 62-66, 2004).

Warning

VistaScan image plate cleaning wipes are classified as highly flammable and irritant according to the Directives relating to the classification, packaging and labeling of dangerous substances and preparations. Avoid contact with the eyes. Keep away from any source of ignition. Do not smoke.

Independent reports – in-house tests

The reports are available on request.