Cod. D4260010
Rev. 00

PROMED

Medical Equipment

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INSTRUCTIONS FOR INSTALLATION USE
AND MAINTENANCE

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ADVICE ON PRODUCT USE

Surface cleaning

Sanitizing liquid for plastics is recommended for surface cleaning.
The chair upholstery should be cleaned with water and neuter soap.

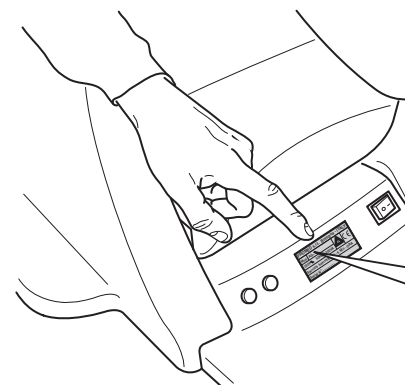
Do not use alcohol-based products.

Disposing

Comply with all applicable local laws when disposing of the equipment and its parts.

Equipment identification

For any communication with the manufacturer or aftersales centres, always mention the equipment's serial number.

PROMED s.r.l. Bologna - ITALY	
Mod. AG 03...	 
230V~ 50Hz 200VA	
Servizio Intermittente - 6 Min - ORA	
Serial N°.....	

Setting of the LP key working mode

Start the chair pressing simultaneously the **LP** key and a key of the movement pushbutton (**1** previous page) according to the selected mode.

Keep the two keys pressed for at least **2** seconds after the starting.

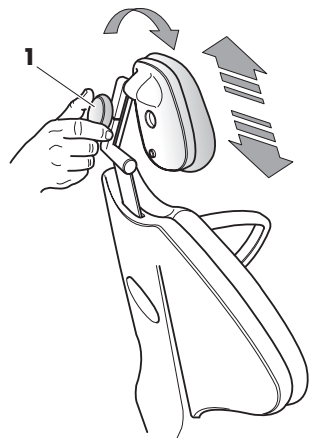
The function of the **LP** key has been stored:

For **mode 1** - Press **LP** + Up

For **mode 2** - Press **LP** + Down


For **mode 3** - Press **LP** + Backrest up

For **mode 4** - Press **LP** + Backrest down

Headrest movement

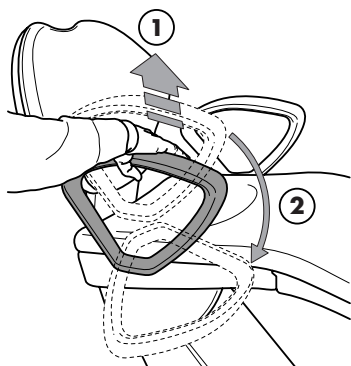The headrest can be more or less extracted from the backrest only by pulling or pressing it because it is automatically blocked by a clutch. The double joint for wide articulation can be adjusted by the knob (**1**): Rotating it anticlockwise it is unblocked, clockwise it is blocked.

Right arm movement

The right arm can be lowered towards the chair front side to facilitate the patient's access. For doing it, first of all pull it upwards (**1**) and then turn it forwards and downwards (**2**). For restoring the normal position carry out the reverse operation.

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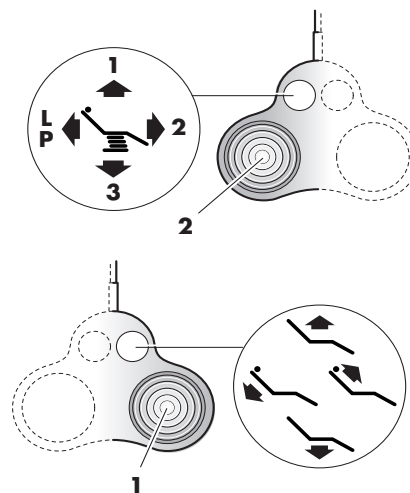
USE (follows)

Automatic controls of the programs

They start pressing with an impulse the left head (**2**) of the double foot-control in one of the **4** indicated directions. The **3** programmable positions correspond to the **1-2-3** directions.

The function of the **LP** key can be set in **4** different working modes.

Mode 1 (Last Position): Starting the **LP** key, the chair comes back to the previous position.

Mode 2 (4th program): The **LP** key has the function of 4th programmable position.

Mode 3 (rinse position for units fastened to the chair): Starting the **LP** key, the backrest is straightened automatically for bringing the patient to the rinse position.

Starting it a second time, the backrest comes back to the previous working position.

Mode 4 (rinse position for units fastened to the floor): Starting the **LP** key, the backrest is straightened automatically and the movement of the seat begins bringing the patient to the rinse position.

Starting it a second time, the backrest comes back to the previous working position.

Warning!

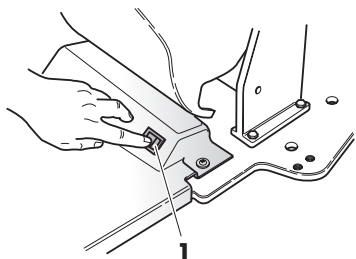
The programmed movements can be stopped pressing the key (**1**) in any one of the **4** directions.

Memorization of the programs

For changing a stored position as well as the possible rinse position, bring the chair to the position to be stored then press the left head (**2**) for at least **10** seconds in the position to be stored. The previously stored position is immediately replaced with the new one.


Notice for the user!

Carry out all the cleaning and adjustment operations as indicated.

Equipment switching on

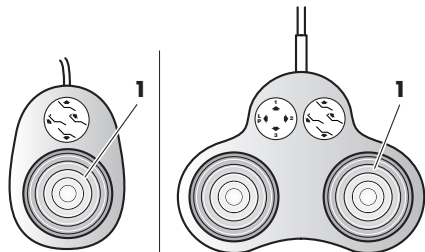The equipment is switched on by setting in the main switch (1) in "I" position. The green light indicates that the chair is on.

PATIENT POSITIONING

Driven movements

The movement controls are started by pressing the head (1) of the foot control (simple for chairs with only four movements or double for chairs with reset or programmes) in the direction of the desired movement, which will be activated up to the release of the head itself.

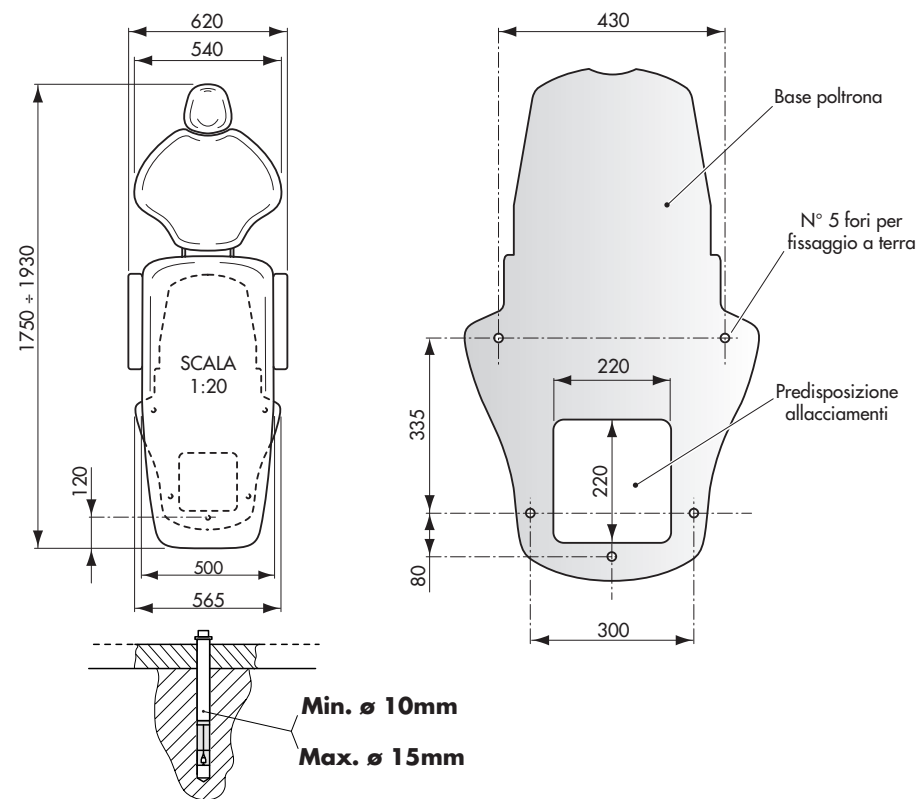
Automatic reset control

It is started by pressing with a pulse the left head (2) of the double foot control in one of the 4 marked directions.


Warning!

The reset movement can be stopped pressing the key 1 in any one of the 4 directions.

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
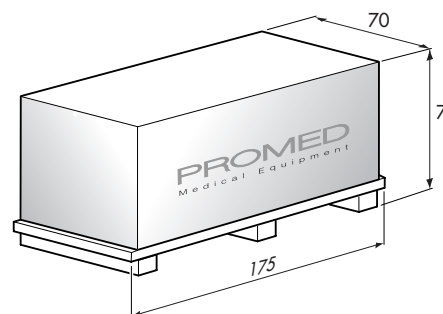
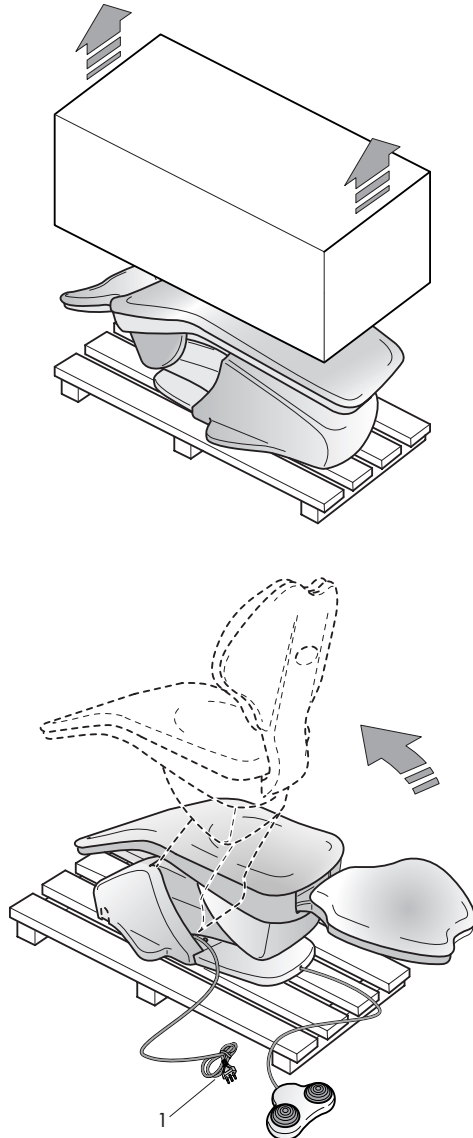
INSTALLATION

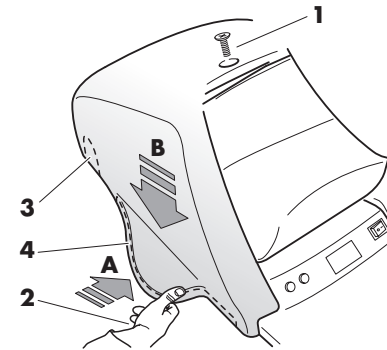8) After the test, assemble the base guard again pressing the point (2) inwards (A) and then downwards (B) to hook it again and fit the screw (1).

If the installation is made by entering the connections from outside, open the pre-arranged opening (3) completely.

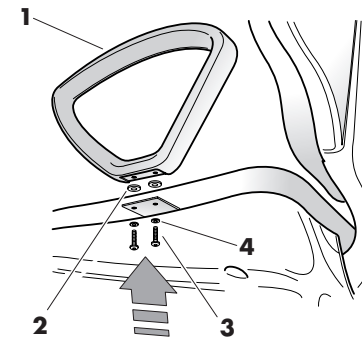If the installation involves the assembly of the stability kit, open the pre-arranged opening (4) completely.


Left arm assembly

Fit the arm (1) on the proper bracket after having placed 2 wide washers (2). Then tighten completely the screws (3) interposing the washers (4).


Warning!

For packing reasons, the arm is assembled downwards.

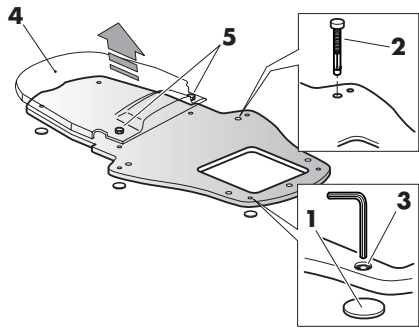
INSTALLATION (follows)

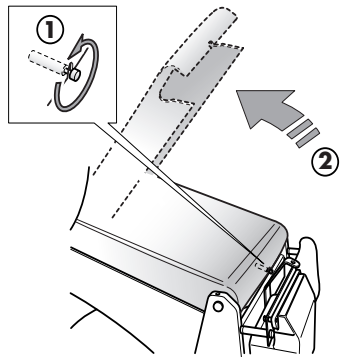
5) Place the 6 stainless steel disks **(1)** supplied under the levelling screws and fasten the base to the ground with the expansion stoppers **(2)** according to the kind of material of which the floor is made, using at least four of the five holes pre-arranged. Fasten the expansion stoppers **(2)** and, if necessary, level the base with the screws **(3)** taking care they exert pressure on the stainless steel disks. If necessary, act on the rear screws and remove the guard **(4)**, unscrewing the screws **(5)** and pulling it upwards.

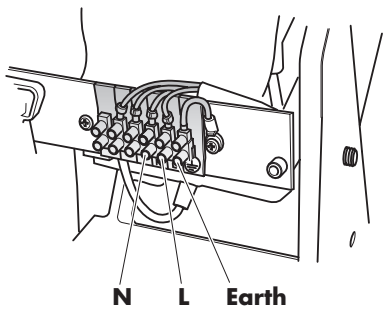
6) For entering the chair arm and for passing the water unit connections, loosen the screw **(1)**, pull the guard upwards **(2)** and then forwards until it is completely pulled out taking care of placing the chair at a proper height. For installing it again carry out the reverse operation.

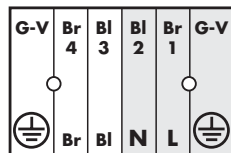
7) Carry out all the installation operations for the water unit and the electrical connection of the chair connecting the power mains wires to the terminal board to **N - L** and **Earth** terminals and carry out a complete test of the equipment.

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
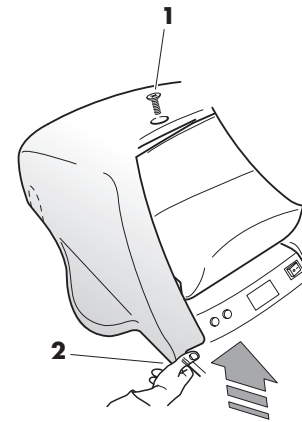
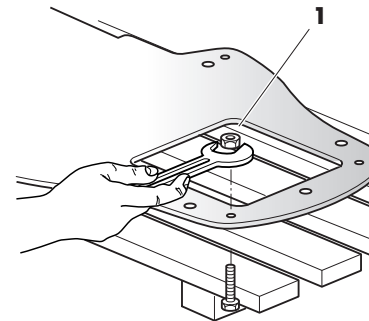
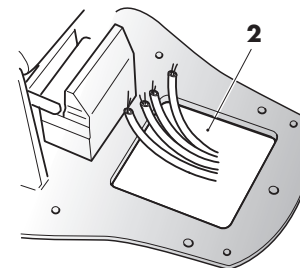
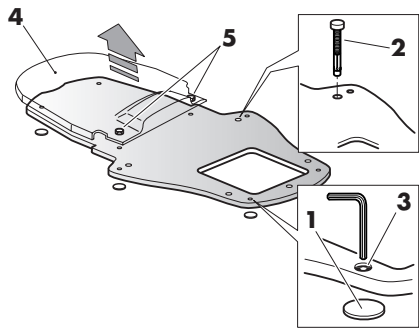
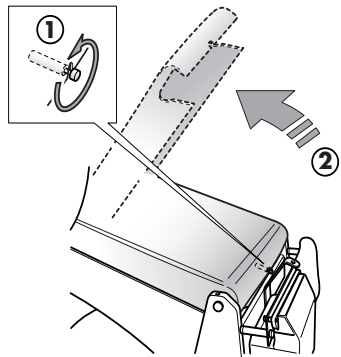
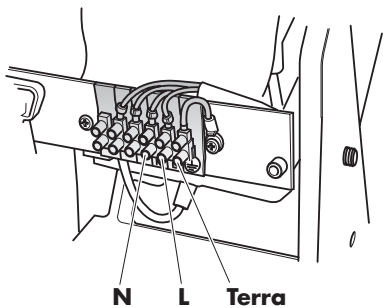
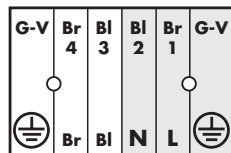
INSTALLATION (follows)

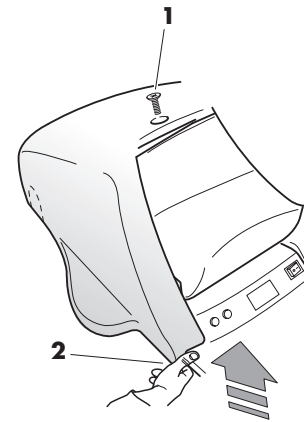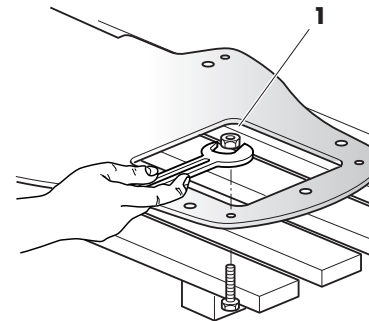3) Unscrew the screw (1) fastening the base guard and remove it pulling the point (2) upwards for overcoming the resistance of the fastening hooks.

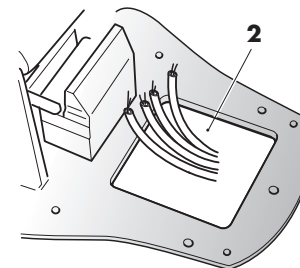4) Unscrew the nuts (1) fastening the base to the pallet and place the chair in the point pre-arranged for the installation taking care of passing the different connecting cables from the floor level through the proper opening of the base (2).

INSTALLATION (follows)

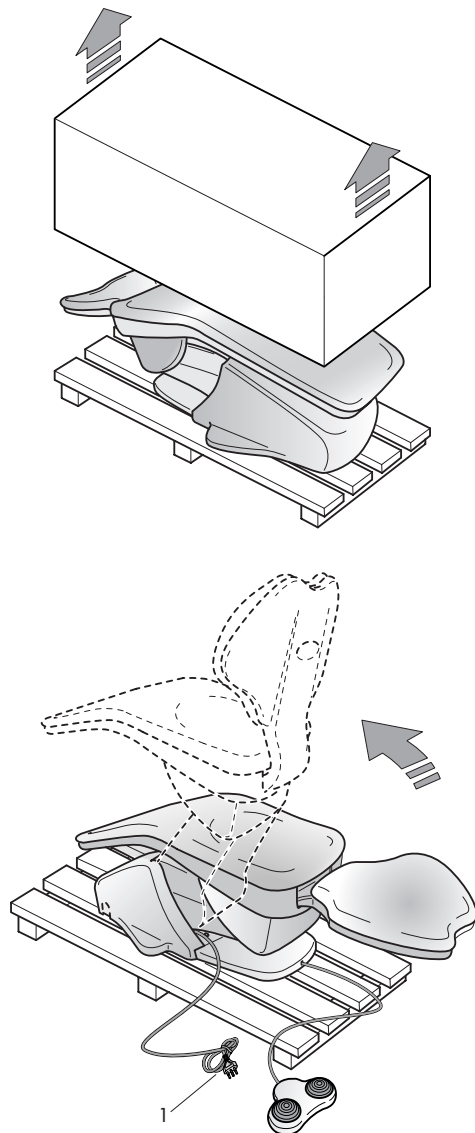
Notice for the installer!

The system should be installed in order to allow the user the best result.


Preparation

1) Remove the upper and side parts of the packing leaving the chair assembled on the pallet in order that it can be transported more easily. Remove the protections of the chair and all the fittings contained in the packing itself.

2) Connect the feeding cable (1) of the chair, coming out from the base, to a socket. Switch on the main switch and, by means of the foot controls, start the forward movement to the maximum height just as the backrest movement until the backrest is tilted up. After this operation **disconnect the feeding cable**.

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
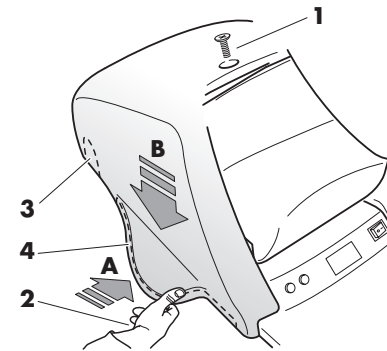
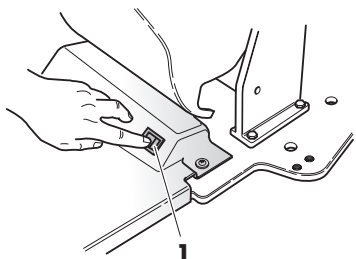
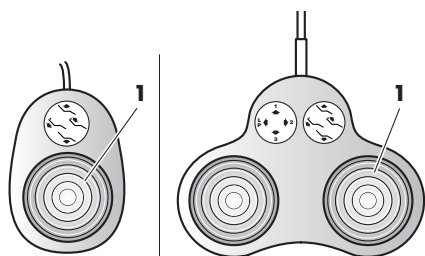
Installation

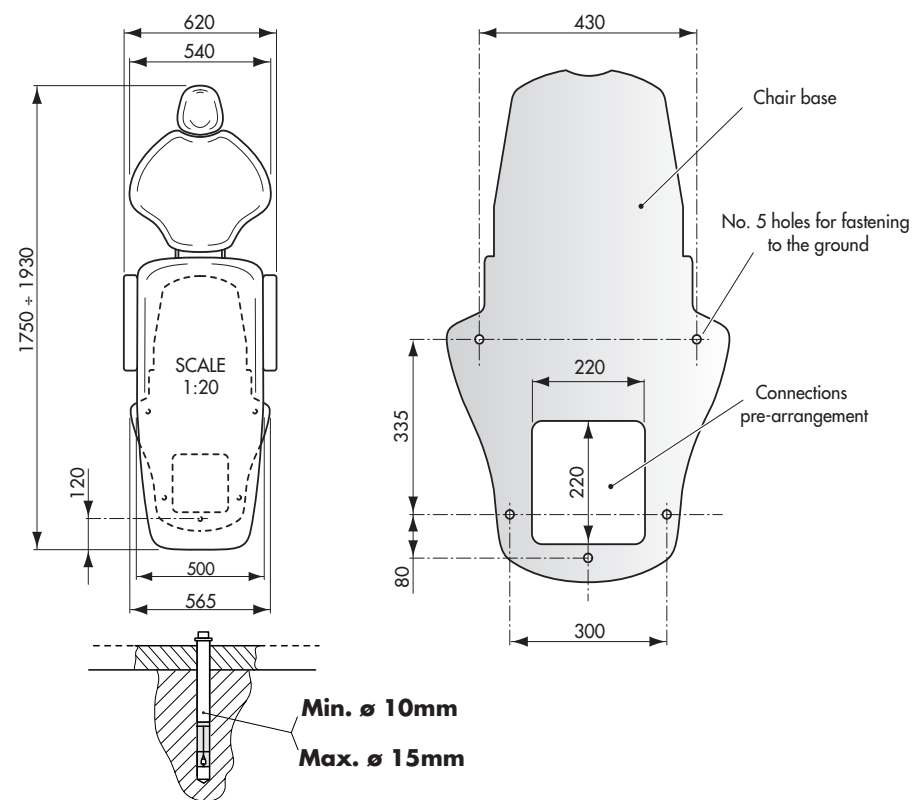Packing

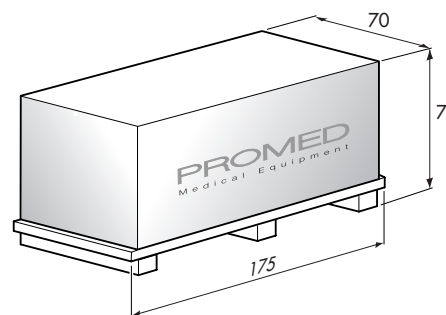The equipment is delivered completely assembled and packed in one box.

Dimension = 175 x 70 x 70 cm.
 Net weight = 100 Kg.
 Gross weight = 120 Kg.
 Volume = 0,86 m³.

PRE-REQUIREMENTS

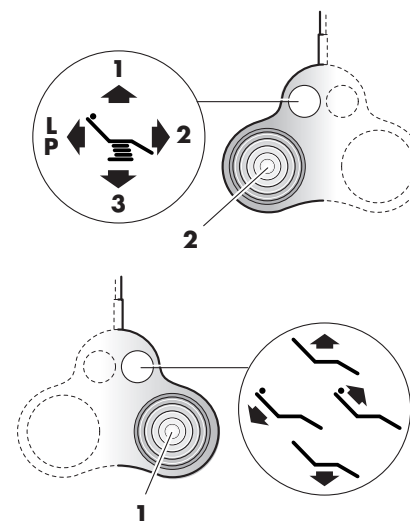
For a right and safe installation of the equipment, the following prescriptions should be complied with:

- The floor should comply with **DIN 1055 B 1.3** and **DIN 18560 T 1** rules.
- An outer bipolar switch should be assembled complying with the law and with following electrical characteristics: **250V / 10A** for disconnecting the equipment in case of intervention in it.
- The electrical system should comply with **IEC 64.4** Regulations. It consists of differential protective conductors (with index **<30mA**) at **250V / 16A**.
- Mobile multiple sockets or extension cords should not be connected to the system.
- Do not connect devices which are not properly indicated as a part of the system.
- The earthing, in particular, should comply with the **art. 3.2.01** of the above-mentioned Regulations.
- The possible pre-arrangements should be carried out according to the indications given in the installation plan.
- The fastening to the ground should be carried out with dowels of **10mm** minimum diameter using at least four of the five holes prepared.

The manufacturer is not responsible for the equipment safety and reliability if:

- **The installation and reparation operations are not carried out by authorized technicians.**
- **The general and electrical installation of the room do not comply with the above-mentioned requirements.**
- **The equipment was not installed complying with this manual.**
- **The equipment was not used complying with this manual.**

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TECHNICAL DATA

TRADE MARK	PROMED
CHAIR MODELS	AG032.../AG033.../AG034...
RATED VOLTAGE	230V~ ± 10%
RATED FREQUENCY	50Hz
RATED POWER	200 VA
OPERATIVE MODE - INTERMITTENT	6 MIN./HOUR
LIFTING CAPACITY	AG032=170 KG. ; AG033=230 KG. ; AG034=300 KG.
LIFTING STROKE	400mm
CLASSIFICATION	CLASS 1

Protecting earth.

Attention, see enclosed documentation.

The equipment complies with the fundamental requirements of **CEE 93/42** Standard.

ENVIRONMENTAL CONDITIONS

Transport and storage environmental conditions

- Environmental temperature from **-40°C** to **+70°C**.
- Relative humidity from **10%** to **90%**.

Operating environmental conditions

- Environmental temperature from **+5°C** to **+50°C**.
- Relative humidity max **80%**.

The equipment does not cause and is not subject to electromagnetic interference.

CONTENTS

	Page
TECHNICAL DATA	16
ENVIRONMENTAL CONDITIONS	16
PRE-REQUIREMENTS	17
INSTALLATION SCHEME AND PACKING	18
INSTALLATION	19
PREPARATION	19
LEFT ARM ASSEMBLY	22
USE	23
EQUIPMENT SWITCHING ON	23
PATIENT POSITIONING:	23
- DRIVEN MOVEMENTS	23
- AUTOMATIC RESET CONTROL	23
- AUTOMATIC CONTROLS OF THE PROGRAMS	24
MEMORIZATION OF THE PROGRAMS	24
SETTING OF THE LP KEY WORKING MODE	25
HEADREST MOVEMENT	25
RIGHT ARM MOVEMENT	25
ADVICE ON PRODUCT USE	26
SURFACE CLEANING	26
DISPOSING	26
PRODUCT IDENTIFICATION	26

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