# PROMED

Medical Equipment

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
---

INSTRUCTIONS FOR INSTALLATION USE  
AND MAINTENANCE

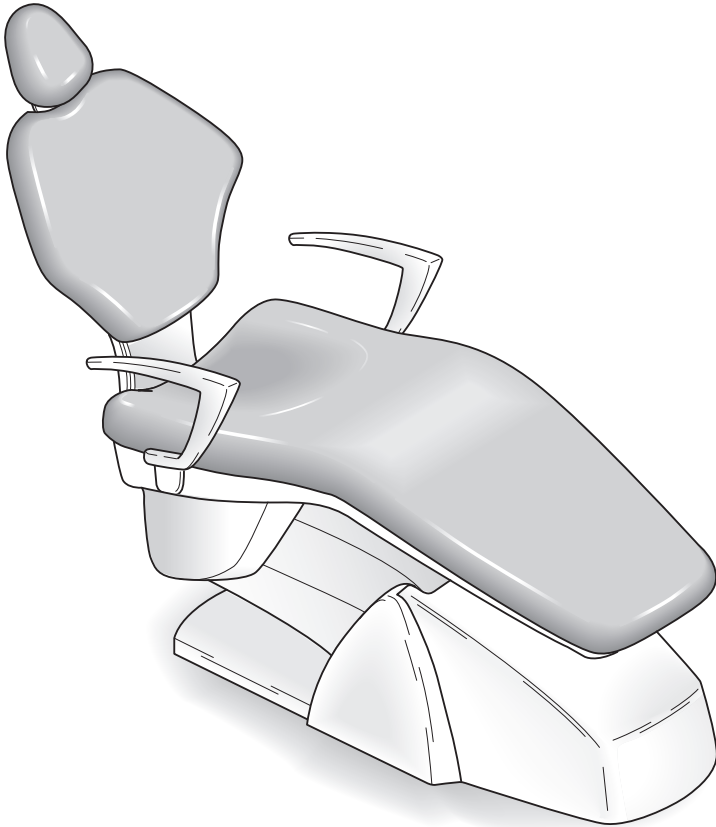CHAIR

AG 04 E

---

# CONTENTS

---

	Page
TECHNICAL DATA .....	17
ENVIRONMENTAL CONDITIONS .....	17
PRE-REQUIREMENTS .....	18
INSTALLATION SCHEME AND PACKING .....	19
<b>INSTALLATION</b> .....	<b>20</b>
PREPARATION .....	20
LEFT ARM ASSEMBLY .....	23
<b>USE</b> .....	<b>24</b>
EQUIPMENT SWITCHING ON .....	24
PATIENT POSITIONING: .....	24
- GUIDED CONTROLS .....	24
- AUTOMATIC RESETTING AT ZERO (for non-programmable chair) .....	24
- PROGRAMMED AUTOMATIC CONTROLS .....	25
MEMORIZATION OF THE PROGRAMS .....	25
SETTING THE FUNCTIONING MODE OF THE LP BUTTON .....	26
HEADREST MOVEMENT .....	26
RIGHT ARM MOVEMENT .....	26
<b>ADVICE ON PRODUCT USE</b> .....	<b>27</b>
SURFACE CLEANING .....	27
DISPOSING .....	27
EQUIPMENT IDENTIFICATION .....	27

## TECHNICAL DATA

---

TRADE MARK	<b>PROMED</b>
CHAIR MODELS	<b>AG042...E/AG044...E</b>
RATED VOLTAGE	<b>230V~ ± 10%</b>
RATED FREQUENCY	<b>50Hz</b>
RATED POWER	<b>200 VA</b>
OPERATIVE MODE - INTERMITTENT	<b>6 MIN./HOUR</b>
LIFTING CAPACITY	<b>AG042E=170 KG. ; AG044E=300 KG.</b>
LIFTING STROKE	<b>400mm</b>
CLASSIFICATION	<b>CLASS 1</b>


Protecting earth.

Attention, see enclosed documentation.

The equipment complies with the fundamental requirements of **CEE 93/42** Standard.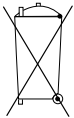

Electric and electronic equipment for civil and professional use that should be disposed of according to specific rules, at the end of its useful life.

## ENVIRONMENTAL CONDITIONS

---

### Transport and storage environmental conditions

- Environmental temperature from **-40°C** to **+70°C**.
- Relative humidity from **10%** to **90%**.

### Operating environmental conditions

- Environmental temperature from **+5°C** to **+50°C**.
- Relative humidity max **80%**.

The equipment does not cause and is not subject to electromagnetic interference.

## PRE-REQUIREMENTS

---

For a right and safe installation of the equipment, the following prescriptions should be complied with:


- The floor should comply with **DIN 1055 B 1.3** and **DIN 18560 T 1** rules.
- An outer bipolar switch should be assembled complying with the law and with following electrical characteristics: **250V / 10A** for disconnecting the equipment in case of intervention in it.
- The electrical system should comply with **IEC 64.4** Regulations. It consists of differential protective conductors (with index **<30mA**) at **250V / 16A**.
- Mobile multiple sockets or extension cords should not be connected to the system.
- Do not connect devices which are not properly indicated as a part of the system.
- The earthing, in particular, should comply with the **art. 3.2.01** of the above-mentioned Regulations.
- The possible pre-arrangements should be carried out according to the indications given in the installation plan.
- The fastening to the ground should be carried out with dowels of **10mm** minimum diameter using at least four of the five holes prepared.

**The manufacturer is not responsible for the equipment safety and reliability if:**


- **The installation and reparation operations are not carried out by authorized technicians.**
- **The general and electrical installation of the room do not comply with the above-mentioned requirements.**
- **The equipment was not installed complying with this manual.**
- **The equipment was not used complying with this manual.**
- **The equipment should be used by personnel qualified in the dental practice for therapeutic purpose.**

# INSTALLATION SCHEME AND PACKING

## Installation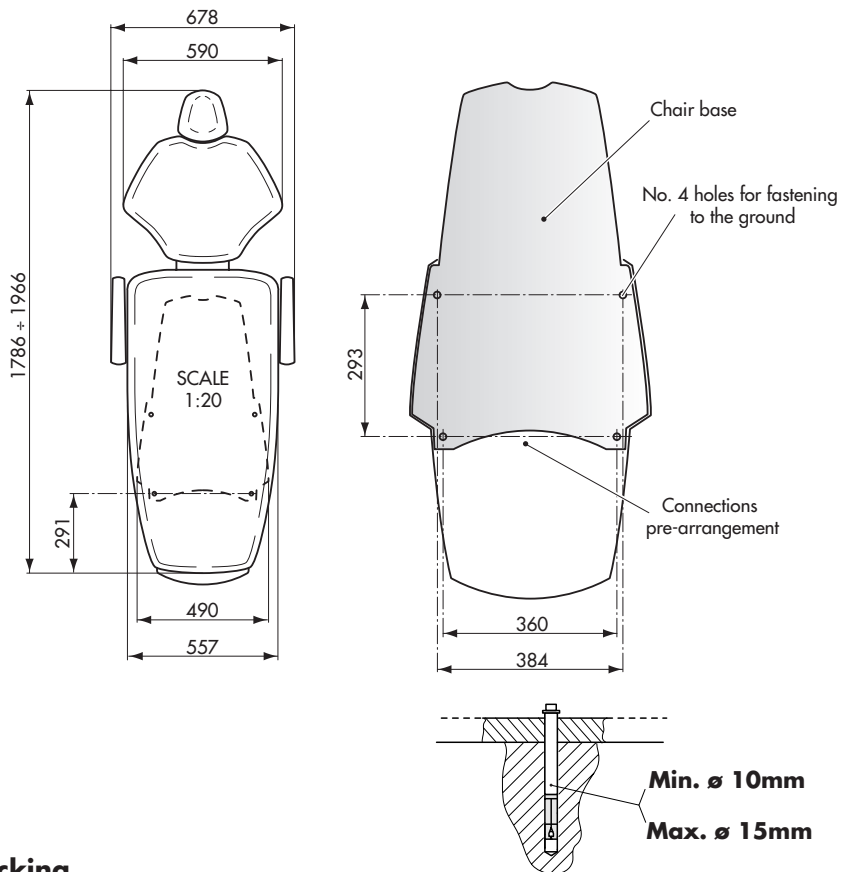

## Packing

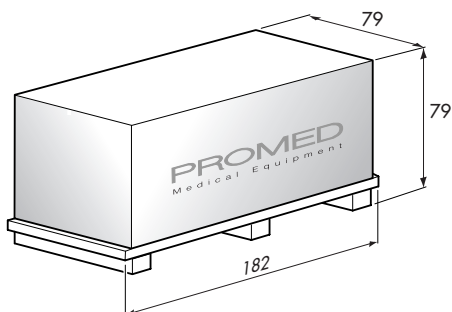The equipment is delivered completely assembled and packed in one box.

Dimension = **182 x 79 x 79 cm.**  
Net weight = **120 Kg.**  
Gross weight = **140 Kg.**  
Volume = **1,14 m<sup>3</sup>.**


## INSTALLATION (follows)

---


**Notice for the installer!**

**The system should be installed in order to allow the user the best result.**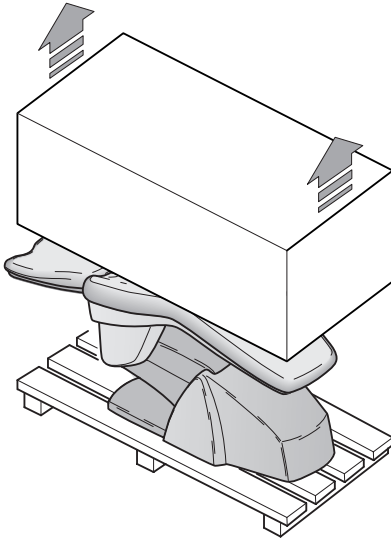

### Preparation

**1)** Remove the upper and side parts of the packing leaving the chair assembled on the pallet in order that it can be transported more easily. Remove the protections of the chair and all the fittings contained in the packing itself.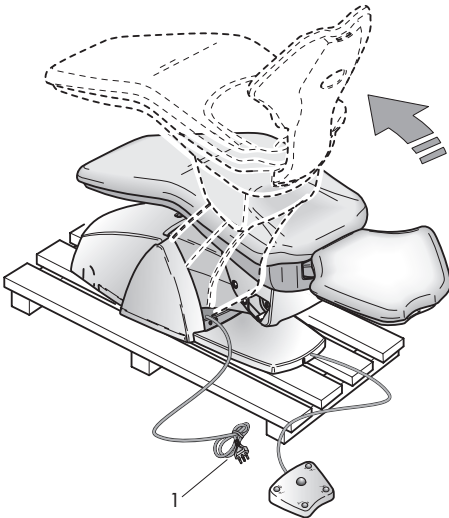


**2)** Connect the feeding cable (**1**) of the chair, coming out from the base, to a socket. Switch on the main switch and, by means of the foot controls, start the forward movement to the maximum height just as the backrest movement until the backrest is tilted up. After this operation **disconnect the feeding cable**.



## INSTALLATION (follows)

---

- 3)** Unscrew the screw (1) fastening the base guard and remove it pulling the point (2) upwards for overcoming the resistance of the fastening hooks.

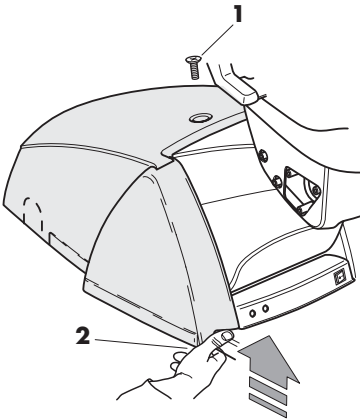
- 4)** Unscrew the nuts (1) fastening the base to the pallet and place the chair in the point pre-arranged for the installation taking care of passing the different connecting cables from the floor level through the proper opening of the base (2).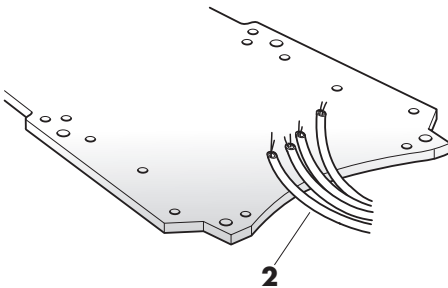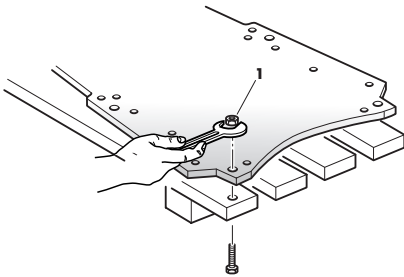


## INSTALLATION (follows)

---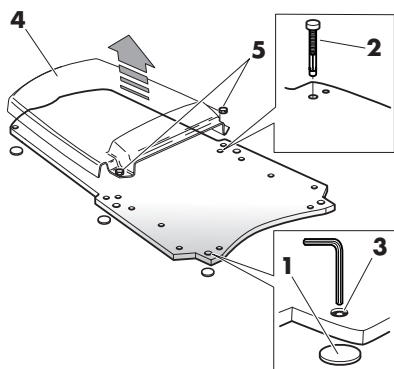


- 5) Place the 6 stainless steel disks (1) supplied under the levelling screws and fasten the base to the ground with the expansion stoppers (2) according to the kind of material of which the floor is made, using at least four of the five holes pre-arranged. Fasten the expansion stoppers (2) and, if necessary, level the base with the screws (3) taking care they exert pressure on the stainless steel disks. If necessary, act on the rear screws and remove the guard (4), unscrewing the screws (5) and pulling it upwards.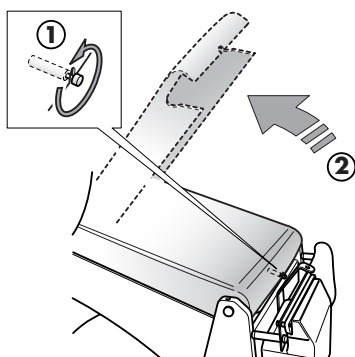


- 6) For entering the chair arm and for passing the water unit connections, loosen the screw (1), pull the guard upwards (2) and then forwards until it is completely pulled out taking care of placing the chair at a proper height. For installing it again carry out the reverse operation.

- 7) Carry out all installations for the water unit and make the electrical connections to the chair. The connector **CN8**, which is normally provided with an electrical jumper, can be linked to a contact from the dental unit which, if open, prevents all chair movements. A keyboard can be connected to the terminal board to control the chair from the dental unit.

# INSTALLATION

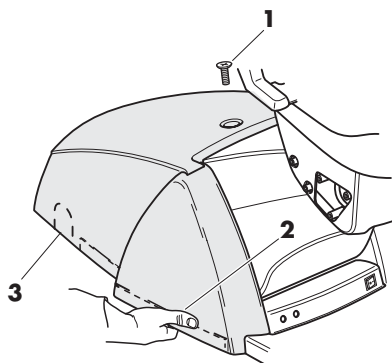- 8)** After the test, assemble the base guard again fasten the screw (**1**).  
If the installation is made by entering the connections from outside, open the pre-arranged opening (**3**) completely.  
If the installation involves the assembly of the stability kit, open the pre-arranged opening (**2**) completely.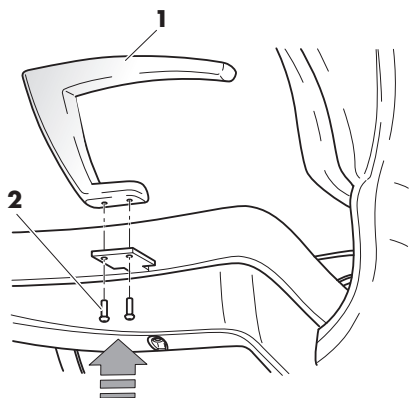

## Left arm assembly

Fit the arm (**1**) on the proper bracket and tighten completely the screws (**2**).

### Warning!

For packing reasons, the arm is assembled downwards.


## USE (follows)

---


**Notice for the user!**

**Carry out all the cleaning and adjustment operations as indicated.**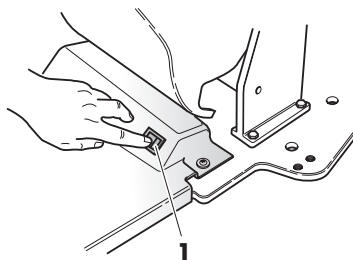

### Equipment switching on

The equipment is switched on by setting in the main switch (1) in "I" position. The green light indicates that the chair is on.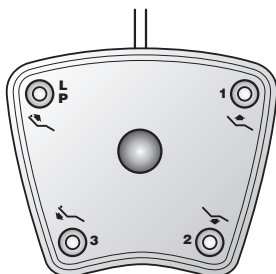

## PATIENT POSITIONING

### Guided controls

These are activated by pressing the button on the foot-control board corresponding to the movement desired. The movement will take place until the button is released.

### Automatic resetting at zero (for non-programmable chair)

This is activated by briefly pressing any one of the foot-control board buttons.


### Warning!

The movement for resetting at zero can be interrupted by pressing any one of the foot-control board buttons again.

## USE (follows)

---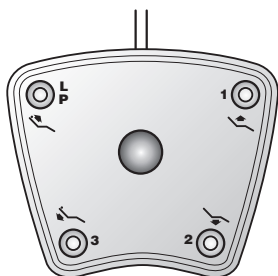

### Programmed automatic controls

These are activated by briefly pressing the foot-control board button corresponding to the programme desired.

Buttons **1-2-3** correspond to three programmable positions.

The function of the **LP** button can be set in **two** different modes.

#### Mode 1

Rinsing position for dental units fitted to the chair.

By briefly pressing the **LP** button, the back of the chair will automatically straighten to bring the patient to the rinsing position.

By briefly pressing it a second time, the back of the chair will return to the previous working position.

#### Mode 2

Rinsing position for floor-fitted dental units.

By briefly pressing the **LP** button, the back of the chair will automatically straighten, and the seat will automatically move to bring the patient to the rinsing position.

By briefly pressing it a second time, the chair will return to the previous working position.


### Warning!

The programmed movements can be interrupted by pressing any one of the pedal-board buttons again.

### Memorization of the programs

To change the position memorised in one of the four programmable buttons (programme no. 1, 2, 3 and LP), move the chair to the desired position with the guided controls; press the lower casing of the chair arm upwards and at the same time press the button in which you wish to memorise the new position until the acoustic signal indicating registration is heard.

The previously memorised position will be replaced by the new position.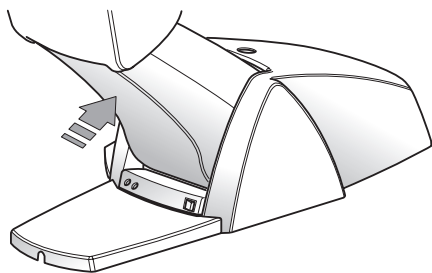

## Setting the functioning mode of the LP button


### Mode 1

To set mode 1, the chair must first be switched off, then jumper **JP1** of the electronic card must be removed.

### Mode 2


To set mode 2, jumper **JP1** must be installed.

**The chair is normally supplied in mode 2.**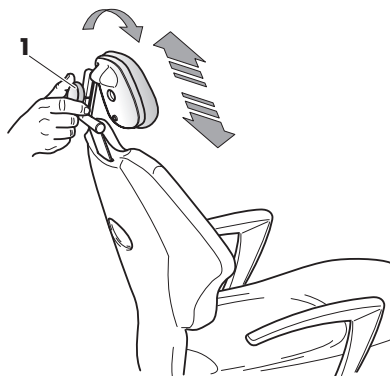

## Headrest movement

The headrest can be more or less extracted from the backrest only by pulling or pressing it because it is automatically blocked by a clutch. The double joint for wide articulation can be adjusted by the knob **(1)**: rotating it anticlockwise it is unblocked, clockwise it is blocked.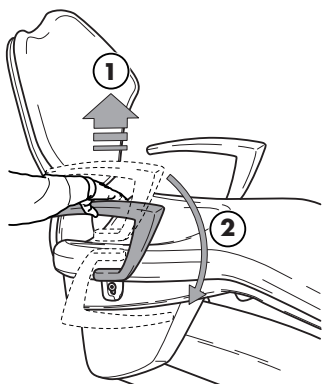

## Right arm movement

The right arm can be lowered towards the chair front side to facilitate the patient's access. For doing it, first of all pull it upwards **(1)** and then turn it forwards and downwards **(2)**. For restoring the normal position carry out the reverse operation.

### Surface cleaning

Sanitizing liquid for plastics is recommended for surface cleaning.

The chair upholstery should be cleaned with water and neuter soap.


**Do not use alcohol-based products.**

### Disposing

At the end of its useful life, estimated by the manufacturer in 10 years from the date of installation, the equipment should be made out-of-service and disposed of at equipped private or public dumping grounds for the disposal of WEEC (Waste Electric and Electronic Equipment).

### Equipment identification

For any communication with the manufacturer or aftersales centres, always mention the equipment's serial number.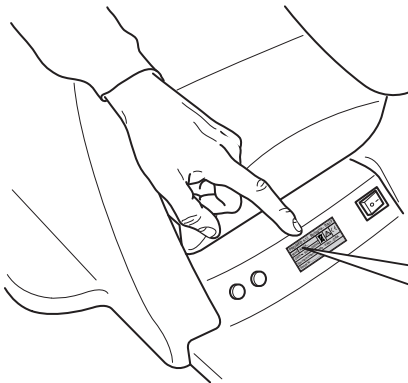

<b>PROMED s.r.l. Bologna - ITALY</b>			
Mod. AG 04...			
230V~ 50Hz 200VA			
Servizio Intermittente - 6 Min - ORA			
Serial N° .....			

Cod. D4260250  
Rev. 00

**PROMED** s.r.l.  
Medical Equipment

Via Grandi 7/2 - 40050 Villanova di Castenaso (BO) - Italy  
Tel. + 39-051-780 465  
Fax. + 39-051-781 625  
E-mail: [sales@promeditaly.it](mailto:sales@promeditaly.it)  
[www.promeditaly.it](http://www.promeditaly.it)