PROMED

Medical Equipment

Cod. D4260040
Rev. 02

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L'USO

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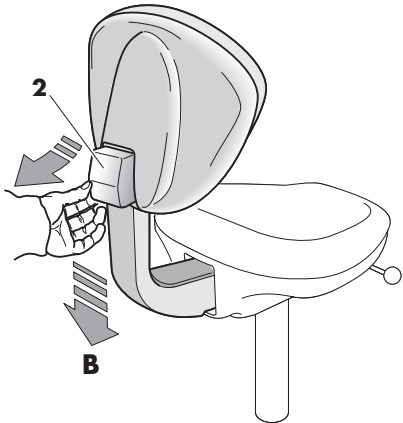
Adjustment of the height of the backrest

- To adjust the height of the backrest (A; B), and provide the optimum support in all operating conditions, push lever (2) along the direction shown by the arrow in the figure.

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MODELS

AG 70020/AG 70021

Caution, see enclosed documentation.

The product complies with the requirements of EN ISO 7493 Standard.

ADVICES ON PRODUCT USE

Surface cleaning

Sanitizing liquid for plastics is recommended for surface cleaning. The stool upholstery must be cleaned with water and neutre soap.

Do not use alcohol-based products.

Disposing


Comply with all applicable local laws when disposing of the equipment and its parts.

Product identification

For any notification about your equipment directly to the manufacturer or to the Aftersales-Centres, please mention the equipment's serial number.

PROMED s.r.l. Bologna - ITALY		 
Mod.		
AG 7002..		
Serial N°	

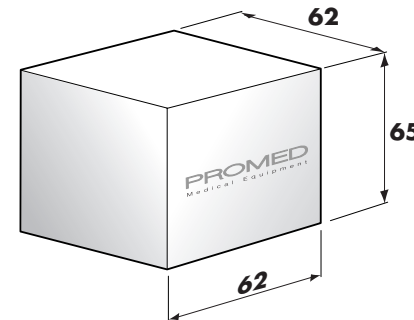Package

The equipment is contained into one packing case.

Dimension = **62 x 62 x 65 cm.**
 Net weight = **14,5 Kg.**
 Gross weight = **16 Kg.**
 Volume = **0,25 m³.**

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
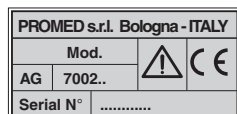
Ergonomical adjustment of the backrest

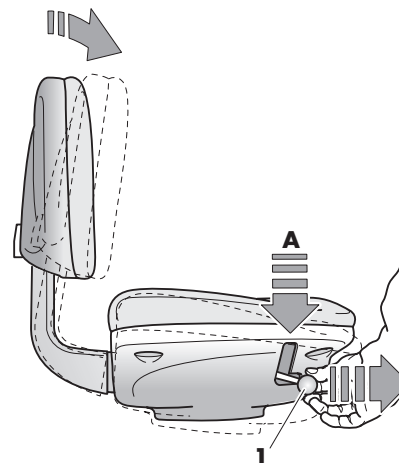- By activating control lever (1) downwards first (A) and forwards then (B), the inclination motion of the backrest is loose and the backrest can therefore constantly fit to the longitudinal motions of the operator's bust.
- To resume the position, push lever (1) backwards.

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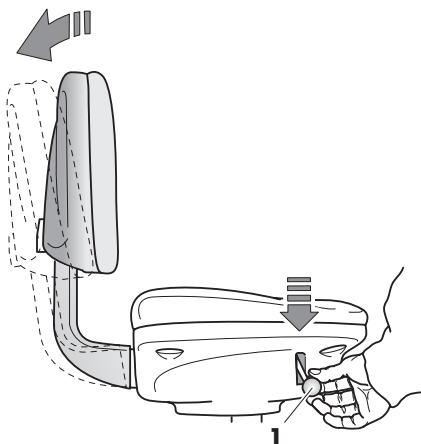
Inclination of the backrest

- To tip the backrest of the operator stool, push lever (1) downwards up to its limit. Once the optimum position is reached, release lever (1).
- To resume position push lever (1) downwards up to its limit.

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Assembly

- Lean base (1) on the floor.
- Place post (2) into its housing on the base.
- Seize firmly the upper portion of the operator stool (3) and locate it on post (2).

Sit down a few times, so as to assure that parts perfectly couple.

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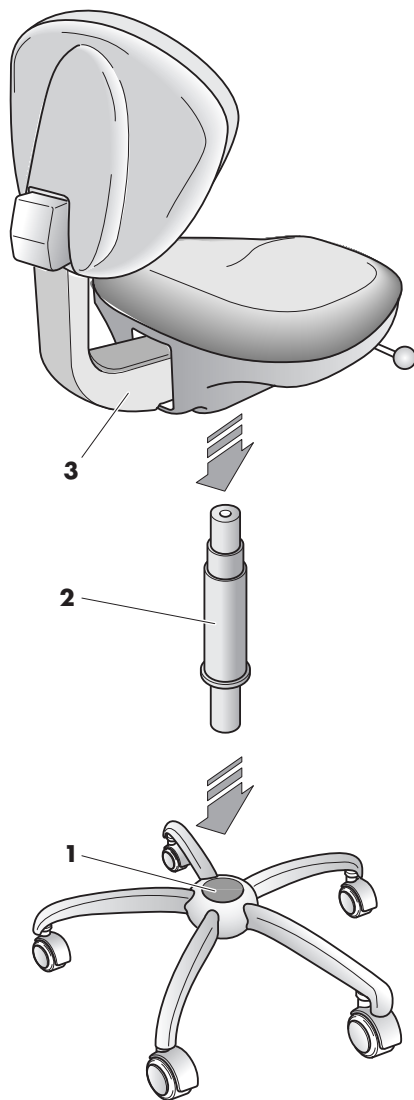
Up and down motions

- To lower the operator stool, keep staying sit down and activate control lever (1) upwards.
- To rise the operator stool, slightly get up from the seat and activate lever (1) upwards.

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